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VENOray

Instructions for Use

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1 Intended Use

The Venoray is a portable LED light source for locating blood vessels in the extremities of premature babies, infants and young children.

2 Basic information

The Venoray is made from polyamide and contains no Latex, DEHP or BPA.

Clean with a mild disinfectant, following the cleaning instructions of the disinfectant manufacturer.

Requires 2 x AAA batteries

Check for damage and proper functionality before use.

Only for clinical use.

Only use on extremities.

Not for prolonged contact with the skin.

Do not look directly into the light source or direct it at the patient's head. Note: this can lead to impaired vision.

If serious incidents occur in connection with the product, please report these to the manufacturer and the responsible authorities.

Non-sterile medical device.

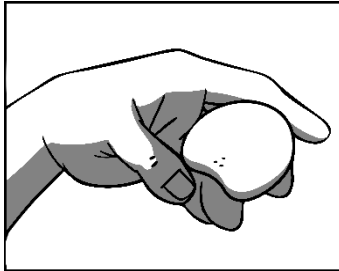
Store in a dry location.

The Venoray is made from polyamide and contains no Latex, DEHP or BPA.

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3

Application

Place the infant's hand on the outside edge of the Venoray, over the LEDs.

The fingers can then be secured over the edge.

Tighten the skin on the back of the hand using your thumb and forefinger.

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4

Disposal

Dispose of used batteries according to the manufacturer's instructions.

If the device is defective, dispose of it as clinical waste.