TC_{matt} NEO_{multi}

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1 Intended Use

The mattress is used for full body temperature control of patients, principally newborns and infants. It connects to the TECOthem NEO via the hose set and temperature regulated liquid flows through the system. In order to improve the temperature control of the patient, the mattress can be wrapped around the body if necessary and closed with securing straps on the eyelets.

2 Application

The mattress must only be used as an accessory of the TECOthem NEO hypothermia system!

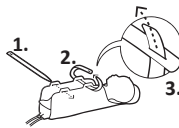
The mattress has to be cleaned before and after each use with a mild disinfectant.

Follow the instructions for use of the TECOthem NEO for correct use of the reusable mattress!

3 Label

TC _{matt} NEO _{multi} Reusable Aqua Wrap (Multiple Use)		EN	Tec Com Medizintechnik
<p>Hygiene: Non-sterile medical device.</p> <p>Cleaning: The mattress must be cleaned before and after each use with a mild disinfectant!</p> <p>Application: Only for use with the TECOthem NEO! Follow the Instructions for Use of the TECOthem NEO.  Direct contact of the mattresses with the patient's skin should be avoided, especially in the case of newborns. Use a water repellent clinically clean drape on the mattress. Risk of damage from sharp objects!</p> <p>Storage: Store in a dry and dark place at room temperature.</p> <p>Disposal: Dispose as clinical waste after expiry of the shelf life.</p> <p>Positioning:</p> 	<p>Material: Polyurethane</p> <p>Inlet/Outlet tubes: Polyurethane</p> <p>Couplings: Acetal</p> <p>Maximum pressure: 500 mbar</p> <p>Size: 42 cm x 62 cm</p>	<p>Distributor:</p> 	
   			

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Tec Com Medizintechnik GmbH