**Tec Com**  
Medizintechnik

**Gebrauchsanweisung**  
**Instructions for Use**  
**Инструкции за употреба**  
**Návod k použití**  
**Brugsanvisning**  
**Οδηγίες χρήσης**  
**Instrucciones de uso**  
**Kasutusjuhend**  
**Käyttöohje**  
**Mode d'emploi**  
**Upute za uporabu**  
**Használati utasítása**  
**Istruzioni per l'uso**  
**Naudojimo instrukcija**  
**Lietošanas instrukcija**  
**Gebruiksaanwijzing**  
**Bruksanvisning**  
**Instrukcja użytkowania**  
**Instruções de utilização**  
**Instrucțiuni de utilizare**  
**Návod na použitie**  
**Navodila za uporabo**  
**Bruksanvisning**  
**Kullanma Kılavuzu**  
**دليل الاستخدام**

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## 1 Intended Use

The TECOthem Wrabbit is designed as a disposable device to help prevent postnatal heat and water loss in premature and newborn infants.

## 2 Basic information

The product is made of PUR film, is latex-free, DEHP-free and BPA-free with a fleece lining.

Product sizes based on the patient's weight:

S < 1000 g; M = 1000 g – 2000 g; L > 2000 g  
Failure to follow these instructions may result in impaired performance.

Only for clinical use.

Do not reprocess or reuse.

Do not use in direct contact with open wounds or skin that is not intact.

Danger of obstructing the airways.

Verify the medical device is correctly fitted.

Keep the airways clear.

Use may result in hyperthermia.

Monitor the patient!

Do not use the medical device to carry the patient.

If serious incidents occur in connection with the product, please report these to the manufacturer and the responsible authorities.

Non-sterile medical device.

For single use only.

Check for damage before use, discard if packaging is damaged.

Keep away from sharp objects, as there is a risk of damage.

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## 3 Application

Step 1:

Lay out the product.  
Place the newborn on the fleece with their shoulders below the shoulder seam.

Step 2:

Fold the hood over the head and bring the sides down along the face.  
Take care to leave the neck region free.

Step 3:

Fold the outer sides over one after the other.  
Swaddle the newborn baby if necessary.  
Access to the patient is possible at any time without damaging the product.  
Tubes can be fed out before folding the outer sides over the end of the feet.

An indwelling venous cannula can be accessed at any time by opening the product on one side, without affecting the patient's thermal insulation.

## 4 Disposal

Dispose of as clinical waste after use.

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[www.service.tcm.gmbh/ifus/](http://www.service.tcm.gmbh/ifus/)

The instructions for use in your native language can be found online at:

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Your relevant distributor can be found online at:

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