

TCmatt NEOsingle **TCmatt NEOmulti**



Instructions for Use

English

Instructions for Use

TCmatt NEOsingle / Disposable Aqua Wrap

TCmatt NEOmulti / Reusable Mattress

Purpose

The mattress is used for the full-body temperature control of patients, especially newborns and infants. It is connected to the TECOtherm® Hypo-/Hyperthermia System through the hose fitting and is filled with tempering fluid. In order to better improve the temperature control of the patient, a mat can be wrapped around the body and secured with straps at the eyelets.

Warnings and Precautions



Only for clinical use.



If serious incidents occur in connection with the product, please report these to the manufacturer and the responsible authorities.



The mattress must only be used as an accessory of the TECOtherm® Hypo-/Hyperthermia System!



This mattress is the only mattress approved by the manufacturer of the TECOtherm® Hypo-/Hyperthermia System.



Especially in the case of newborns, direct contact between the mat and the patient's skin should be avoided. A water-repellent, sterile cover sheet should be used on the mat.



Please follow the instructions for proper cleaning of the product detailed in chapter Cleaning.



Follow the instructions for use of the TECOtherm® Hypo-/Hyperthermia System for correct use of the mat!



Keep away from sharp objects, as there is a risk of damage.



Non-sterile medical device.



This mattress is a single-use product! It must not be reprocessed or used a second time!



Check for damage before use, discard if packaging is damaged.

Symbols



Medical device



Protect from sunlight



Manufacturer



Batch designation



Use by



Catalogue number




Store in a dry location



Store at room temperature

Application

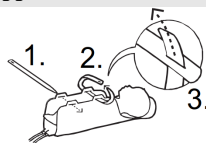

1. Place the child in the centre of the mattress. Pass the straps on both sides through the eyelets provided in the mat.
2. Pull both ends of the straps together so that the mattress closes around the child.
3. Push the narrow, tapered side of the straps through the eyelets on at the other end of the straps. Pull the tips all the way through until the mattress is closed and secured.

Cleaning

Hygiene and processing regulations must be adhered to in accordance with national and institution-specific guidelines.

Nationally approved surface disinfectants with broad-spectrum antimicrobial action suitable for cleaning must be used.

Before and after each use, clean the mattress and hose connections using a lint-free cloth and a gentle, colourless, alcohol-based surface disinfectant with broad-spectrum antimicrobial action (e.g., Bacillol) in accordance with the manufacturer's instructions. Remove any coarse dirt beforehand. If any dirt remains, repeat the cleaning process.

Do not clean mattress, single use only!

Storage

Store in a dry and dark place at room temperature.

Disposal

After expiry, dispose as clinical waste.

Single patient use, dispose as clinical waste.

Product specifications

Material:	Polyurethane
	Polyurethane / Polyester
Inlet/Outlet tubes:	Polyurethane
Couplings:	Acetal
Maximum pressure:	500 mbar
Size:	44 cm x 72 cm
Item number and product name:	642 68 01 / TCmatt NEOmulti
	642 74 01 / TCmatt NEOsingle
Weight:	0.21 kg



The instructions for use in your native language can be found online at: www.service.tcm.gmbh/ifu/



Distributor
LOGO

Ihren zuständigen Distributor finden sie Online unter:

Your relevant distributor can be found online at:

Вашия дистрибутор можете да намерите онлайн на:

Vašeho distributora najdete online na adrese:

Du finder din distributør online under:

Τον αρμόδιο διανομέα σας θα τον βρείτε στο διαδίκτυο στην ηλεκτρονική διεύθυνση:

Para encontrar su distribuidor online, consulte en:

Oma vastutava turustaja leiuate internetist aadressilt:

Löydät asianomaisen jälleenmyyjän verkosta osoitteesta:

Vous trouverez votre distributeur compétent en allant sur :

Svog odgovornog distributera možete pronaći online na:

Az Ön illetékes forgalmazóját az alábbi internetes oldalon találja:

Il distributore competente per la vostra area è disponibile online all'indirizzo:

Atsakingą gaminio platintoją galite rasti internete adresu:

Jūsu valstī atbildīgais izplatītājs atrodams tīmekļa vietnē:

Uw distributeur vindt u online op:

Du finner din ansvarlige distributør online på:

Swojego dystrybutora mogą Państwo znaleźć w Internecie pod adresem:

Pode encontrar o seu distribuidor online em:

Puteți găsi online distribuitorul responsabil la adresa:

Vášho distribútora nájdete online na adrese:

Svojega pristojnega distributerja najdete na spletu na naslovu:

Du hittar din ansvariga distributör online på:

Yetkili distribütörünüzü online olarak burada bulabilirsiniz:

يمكنك العثور على الموزع المسؤول الخاص بك عبر الإنترنت على:



tcm.gmbh/en/distribution/



Tec Com Medizintechnik GmbH
Am Krümmling 1
06184 Kabelsketal
Deutschland
Tel.: +49 345 120 52 04
www.tcm.gmbh